




## Veterinary Blood Glucose Monitoring System

- Rapid blood glucose monitoring for cats and dogs
- Fast, accurate, and compatible with the PETRACKR® app to easily record, monitor, and report results
- Based on one of the market-leading human blood glucose products

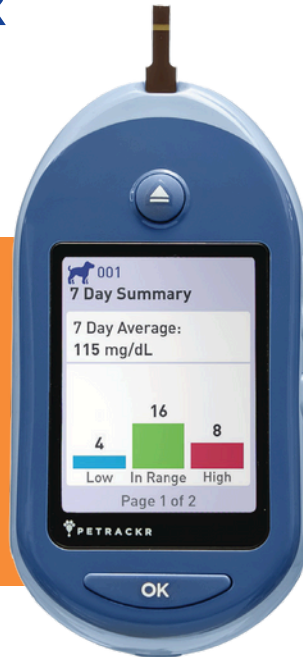



Specifically designed and calibrated for cats and dogs

## About PETRACKR®

Universal Biosensors introduces PETRACKR® Veterinary Blood Glucose Monitoring System, using the same technology that developed the **market-leading human blood glucometer**, now for cats and dogs. The PETRACKR® Blood Glucose Analyzer is small, portable and accurate, **delivering fast and reliable results in seconds**, and easy to use either in the veterinary clinic or at home.

The PETRACKR® Blood Glucose Analyzer and test strips enable pet parents to easily monitor their pet's blood glucose measurements. By connecting seamlessly via **Bluetooth® to the PETRACKR® app**, it creates a pet profile which directly tracks blood glucose measurements.

The PETRACKR® app also supports recording and tracking of a pet's activities including insulin doses, meals and exercise. Pet parents can simply download and **share their pet's results with a veterinarian, allowing easy monitoring and assessment.**

## Why test blood glucose in cats and dogs with diabetes?

The prevalence of diabetes among cats and dogs has increased in recent years. **Human blood glucose monitoring systems are not recommended for blood glucose monitoring in cats and dogs.** The PETRACKR® Veterinary Blood Glucose Monitoring System is **specifically calibrated and designed** for professional monitoring of blood glucose in cats or dogs, **providing confidence that readings are accurate and reliable.**



### Accurate

Compared to laboratory analyzers



### Rapid results

A few seconds to get a result with a small sample size



### Safe to handle

Eject button for sample disposal



### Easy to use

Analyzer and app data collection and management



### No coding

Auto-calibrated for cats and dogs



### Smart phone app

Customizable pet profiles






## PETRACKR® Analyzer

- 1. Test strip port**  
Insert test strip here
- 2. Eject button**  
Eject test strip from analyzer
- 3. Range**  
Indicator
- 4. Last event**  
Date and time stamp saved
- 5. Up and down buttons**  
Highlight the desired selection
- 6. Back button**  
Go back to the previous screen (press and release). Go back to main menu (press and hold)
- 7. OK button**  
Turns analyzer on/off (press and hold). Confirms menu selections (press and release)



Refer to the [PETRACKR® User Guide here](#)

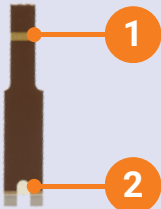

## PETRACKR® Blood Glucose Test Strips

- 1. Sample chamber application area**
- 2. Test strip insertion end**


PETRACKR® Test Strips are specifically designed for use with the PETRACKR® Veterinary Blood Glucose Monitoring System. The test strips are accurate when testing fresh, capillary, or fresh venous whole blood samples. Test strips should be stored in the vial with the lid closed. Keep them in a cool, dry place between 41°F - 86°F (5°C - 30°C) and below 90% relative humidity.

# PETRACKR® app

An essential part of managing your pet's diabetes is regularly monitoring their blood glucose and other lifestyle factors. The PETRACKR® app simply connects to the PETRACKR® Analyzer via Bluetooth allowing you to easily record and track your pet's blood glucose measurements and log important factors such as insulin doses, meals and exercise. The PETRACKR® app is designed to provide flawless and simple reporting of your pet's results and factors for sharing with your veterinarian. The PETRACKR® app works seamlessly with the PETRACKR® Blood Glucose Analyzer to provide human quality care to your furry friend.

## The PETRACKR® app allows users to:

- Seamlessly track your pet's blood glucose measurements
- Create up to five pet profiles linked to your PETRACKR® Blood Glucose analyzer so all blood glucose measurements are transferred to the PETRACKR® app
- Add and track activities including blood glucose measurements, insulin doses, meals and exercise to your pet's logbook
- Easily share your pet's logbook report with your veterinarian
- Set personalized reminders (phone notifications) to stay on top of your pet's diabetic needs
- Colorful charts are available for daily, weekly and monthly periods to identify trends
- View summary statistics of your pet's minimum, maximum and average blood glucose results which is an essential resource for your vet to see
- Enhances the pet parent's ability to promote the pet's overall health
- Encourage the pet parents to share data with the clinic ensuring pet parent responsibility for glucose monitoring

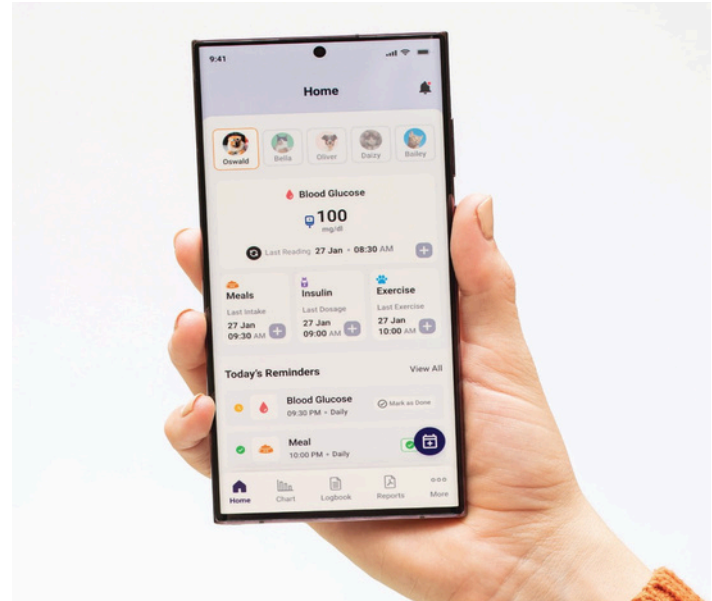



## How to use

### Results in 5 seconds, with 3 simple steps

1. Insert test strip into the strip port of the analyzer
2. Using the PETRACKR® lancing device, get your pet's blood sample by ear or paw. Use your pet's preferred method
3. Apply blood sample to test strip sample chamber



Results displayed in 5 seconds

Item Code	Item Description	Unit of Measure
95002	PETRACKR® Blood Glucose Test Strips	50 strips
95101	PETRACKR® Blood Glucose Monitoring System (mmol/L)	Each
95102	PETRACKR® Blood Glucose Monitoring System (mg/dL)	Each
95202	PETRACKR® Lancets	100 lancets
95302	PETRACKR® Mid Control Solution Kit	Each (1 x 3.8mL)





## About Universal Biosensors

PTRACKR® was developed by Universal Biosensors. Universal Biosensors Inc (UBI) is a global biosensor company and a world leader in electrochemical cell technology. UBI's ambition is to utilize its patented biosensor technology to develop a diverse range of biosensor test strips used in our hand-held portable analyzers; for cost effective, effortless, and accurate detection of analytes of interest, within many industries including human health, animal health, environmental, and agriculture. UBI is a USA Delaware company listed on the Australian Securities Exchange (ASX:UBI) and operates from its head office, manufacturing facility, and research laboratory in Melbourne, Victoria, Australia.

**Placing the universal power of biosensors into the hands of those who need it**



POWERED BY

**Universal**  
Biosensors



PETRACKR® and all related names, logos, product and service names, designs, and slogans are trademarks of Universal Biosensors Pty Ltd and may not be used without the prior written permission of Universal Biosensors Pty Ltd. All other names, logos, product and service names, designs, and slogans are the property of their respective owners.

1 Corporate Avenue, Rowville,  
Victoria, 3178, Australia

© 2024 Universal Biosensors

Inquiries: [PETRACKR@universalbiosensors.com](mailto:PETRACKR@universalbiosensors.com)

Web: [www.universalbiosensors.com/products/petrackr/](http://www.universalbiosensors.com/products/petrackr/)



POWERED BY



**Universal**  
Biosensors